

mamajoo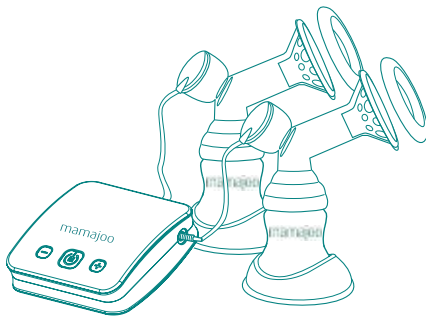

- DE** Mamajoo Elektronische Doppel Milchpumpe mit USB Anschluss
- EN** Mamajoo Electronic USB Double Breast Pump
- FR** Mamajoo Tire-Lait Electronique USB Double
- TR** Mamajoo Elektronik USB Çiftli Göğüs Pompası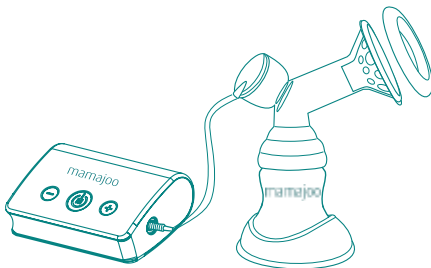

- DE** Mamajoo Elektronische Milchpumpe mit USB Anschluss
- EN** Mamajoo Electronic USB Single Breast Pump
- FR** Mamajoo Tire-Lait Electronique USB Simple
- TR** Mamajoo Elektronik USB Tekli Göğüs Pompası

mamajoo

Index

DE	4
EN	11
FR	18
TR	25

Bedienungsanleitung / Garantie

Instructions / Warranty

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EN Dear Consumer,

Thank you for choosing Mamajoo products. Mamajoo products meet the stringent EU standards as they are designed and produced with highest quality BPA free raw materials combined with latest high technology, by considering health and safety. For more information and your questions, you can find our contact details at the end of this instructions leaflet. Please keep this leaflet for future references.

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ATTENTION

1. Please read all instructions carefully before using.
2. Before first and before & after each use; separate all parts in contact with breast milk, wash properly and sterilise with Mamajoo steam steriliser or boiling them for 5 minutes. Do not sterilise longer than recommended period to avoid damages.
3. Do not expose the motor unit and electrical equipment to liquid and moisture or humidity, do not clean with wet cloth. Never sterilise pump air tube and components which are not in contact with breast milk.
4. Wash the pump's mechanical parts in warm soapy water and rinse or wash in dishwasher top rack.
5. Avoid cleaning with anti-bacterial detergents and abluents that may cause damage or cracking on the product. In such a case please do not use the damaged pieces.
6. Use Mamajoo Electronic USB Breast Pumps with ONLY the original adapter. Check if it turns off correctly after each use and unplug the power cord.
7. Do not exceed the capacity of the breast milk storage / feeding container to avoid motor unit damages. If you get more than 120 ml milk when you pump please use a larger feeding bottle.
8. Keep out of reach of children and babies.
9. Extra hard and fast milk expression may cause pain so that please ensure the power is right for you.
10. If the silicone comfort cushion does not fit correctly to the funnel, the milk flows out from silicone comfort cushion, please check it before each use.
11. When a problem occurs, unplug the power cord or remove the product from the external charger. In case of any requirements of service, please contact us to direct you to our technical customer service.
12. Please be very careful not to pull too strong the stopper (p) of Mamajoo Electronic USB Double Breast Pump.
13. There must be strict compliance with the instructions. Our company will not be held responsible for any damage caused by improper use.

MAINTENANCE

1. Clean the motor unit **ONLY** with a dry cloth. Do not use abrasive or anti-bacterial detergents.
2. Check regularly that the air tube connections are made correctly and the membrane is not damaged.
3. The pump body and the breast milk storage containers can be washed in the dishwasher top rack. However a color change can be occur due to foods.

The tube cannot be sterilized or washed. The water left in the tube might destroy the motor part.

TECHNICAL FEATURES

The advanced intelligent microprocessor technology provides complete control. Processes and signs can be easily monitored from LED indicators. It can be used with an external charger (powerbank) for outdoor? use. External charger must have min. 2600 mA real capacity.

Product Name: Mamajoo Electronic USB Double Breast Pump

Product Code: MMJ2858

Adaptör: Input: 100- 240V ~ 50/60Hz Output: 5-6V 1A DC

Run Time: 0-30 minutes

Weight: 0.170 kg


Product Name: Mamajoo Electronic USB Single Breast Pump

Product Code: MMJ2841

Adaptör: Input: 100- 240V ~ 50/60Hz Output: 5-6V 1A DC


Run Time: 0-30 minutes

Weight: 0.150 kg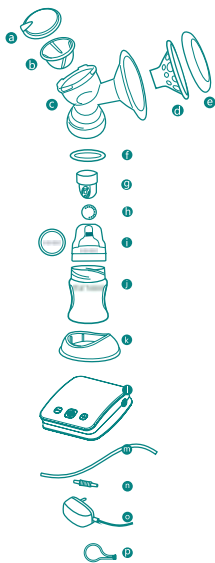

Electronic USB Double Breast Pump product components

- a) Cylinder Cover
- b) Silicone Diaphragm
- c) Pump Body
- d) Silicone Comfort Cushion
- e) Hygiene Cover
- f) Silicone Sealant
- g) Valve
- h) Membrane
- i) Bottle Teats and Sealing Disc
- j) Breasts Milk Storage Container (150 ml)
- k) Storage Container Base
- l) Motor unit
- m) Air Tube
- n) Air Tube – Motor Unit Insert
- o) AC adapter
- p) Stopper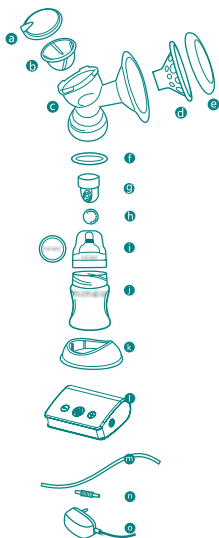

Electronic USB Single Breast Pump product components

- a) Cylinder Cover
- b) Silicone Diaphragm
- c) Pump Body
- d) Silicone Comfort Cushion
- e) Hygiene Cover
- f) Silicone Sealant
- g) Valve
- h) Membrane
- i) Bottle Teats and Sealing Disc
- j) Breasts Milk Storage Container (150 ml)
- k) Storage Container Base
- l) Motor unit
- m) Air Tube
- n) Air Tube – Motor Unit Insert
- o) AC adapter


HOW TO ASSEMBLE

Before starting to assemble, please wash your hands.

Before using wash and sterilize the mechanical pieces of the breast pump.

Snap the membrane onto valve until it lies completely flat.

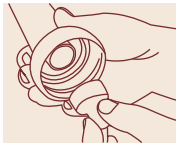Push assembled valve to the pump body tightly.


Place the silicone comfort cushion over the pump body funnel.

Place the silicone diaphragm onto the pump body.

Screw the cylinder cover onto the pump body.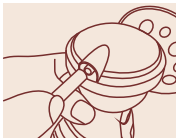


Insert the end of the air tube into the air port of the cylinder cover.

Screw breast milk storage container to the pump body.


Insert the USB plug into the charging source output for mobile use.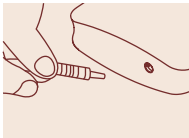

Double Breast Pump

Insert the other end of air tube into the port of the motor unit.

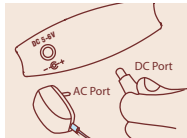
Double Breast Pump

Insert the metal plug at the end of adapter into the adapter input on the motor unit. (5V-6V)
Plug in the adapter. (110V-220V)


Double Breast Pump

Make sure the stopper is firmly connected to the second output for single use. In case of an air escape, pump will not function correctly.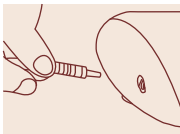

Single Breast Pump

Insert the other end of air tube into the port of the motor unit

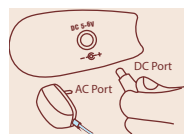Single Breast Pump

Insert the metal plug at the end of adapter into the adapter input on the motor unit.
Plug in the adapter.

LED INDICATOR MANUAL

Mamajoo Electronic USB Double Breast Pump

8 different level options of massage mode

12 different level options of suction mode

Double massage mode
Functions on 3th level


Example

Single suction mode
Functions on 7th level

Control Panel

- a) Single Breast massage/suction mode
- b) Double Breast massage / suction mode
- c) On/Off and Double/Single mode changing button
- d) Suction decrease button
- e) Suction increase button
- f) LED level indicator

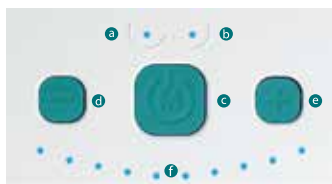Mamajoo Electronic USB Single Breast Pump


5 different massage mode options

10 different suction mode options

Control Panel

- a) Massage position
- b) Suction position
- c) On/Off and massage/suction mode changing button
- d) Suction decrease button
- e) Suction increase button
- f) LED level indicator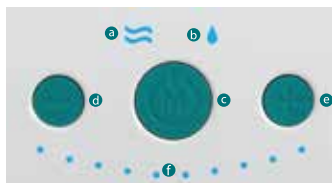

OPERATION

With Adapter:

1. Make sure the breast pump is assembled correctly.
2. For single use, insert the stopper which is located on the motor unit tightly into the second pump air tube input.
3. Sit on a comfortable chair and relax, hold the pump with your hand which you feel comfortable with. Do not use breast pump in leaning position.
4. Center the pump funnel with silicone comfort cushion over your nipple. Make sure it seals your breast completely.
5. Plug in and press 'On/Off' button to start suction. We recommend you to start with massage mode to stimulate your milk flow.
6. Mamajoo Breast Pumps mimic baby's natural suckling rhythm.
 - 6.1. When you press the On/Off button of Mamajoo Electronic USB Single Breast Pump one time, indicator LED at the upper left lights up for the massage mode, if you press the button for a second time indicator LED at the upper right lights up for the suction mode. The massage mode is for to alert the 'let-down' reflex which is like the baby starts to suck the mother's breast. At the first 5 levels for the massage mode; after each 10 short shots a long shot occurs, when your milk starts to flow press the "On/Off" button again and switch the suction mode. You can select the desired expression mode between 1-10 power levels and with "+" or "-" buttons you can adjust suction power. Mamajoo Electronic USB Double/Single Breast Pump saves in memory the last steps you used with its intelligent microprocessor technology. At your next use, the breast pump will start on massage mode and on the last level you used.
 - 6.2. Once you press "On/Off" button, Mamajoo Electronic USB Double/Single Breast Pump starts to run on single mode for offering single use or double use options to the mother. Both for single use or double use first 4 level is for massage mode and the levels from 5 to 10 are for the suction mode. The massage mode is for to alert the 'let-down' reflex which is like the baby starts to suck the mother's breast. At the first 4 levels for massage mode which occurs after each 10 shots short and a long shot when your milk starts to flow press "+" key again and switch the suction mode. You can select the desired expression mode between 5-10 power levels. If you would like to use single suction, make sure that the pump is on single mode. Mamajoo Electronic USB Double/Single Breast Pump, saves in memory the last steps you used with its intelligent microprocessor technology. At your next use, the breast pump will start on single suction mode and on the last level you used.

- When the expression is finished, press “ On/Off” button for 2 seconds and unplug it. Take out the breast milk storage container from the pump. Place the Mamajoo sealing disc on the breast milk container and put into the fridge or the freezer or feed your baby directly.
- Mamajoo Electronic USB Breast Pumps automatically shuts off after it runs 30 minutes with its intelligent micro-processor technology.

External Charger Operation :

In principle, there are not much differences to run the product with an external charger. Instead of plug it with an adapter you can use a fully charged external charger (power-bank, computer etc) Make sure that the external charger power source is correct.(See. Technical Features)

WARNING

The product should be moved by considering the indicated transport direction on the box and by protecting from high temperatures, fire and impacts in its original package during transportation and shipping.

WARRANTY

- This product is under guarantee of our company for 2 years after date of purchase for the first user. Please contact us or the retailer that you purchased the product from for warranty claims.
- This warranty does not cover wears occur from regular use, defects and damages caused by misuse, abuse and improper storage, failure to comply the manual instructions, malfunctions from using improper components.
- The responsibility of the usage the product belongs to the user.
- Please keep your receipt as a proof of purchase for any warranty claim.

Warranty of the product is 2 years starting from the date of original purchase.
Lifetime of the product is 5 years.

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İç ve Dış. Tic. Ltd. Şti.
mamajoo GmbH